



## ADAPTDX PRO POWER MANAGEMENT

### Power supply management for the AdaptDx Pro<sup>®</sup> is vital to consistent trouble-free testing

- The AdaptDx Pro system comes with two handheld controllers
- Each controller has a rechargeable battery which provides power to the headset
- A charged controller must be connected to the headset at all times, even when stored

THE OTHER CONTROLLER SHOULD BE LEFT ON THE WALL CHARGER UNTIL SWAPPED FOR THE DEPLETED ONE ON THE HEADSET

#### Helpful Hint:

- Exchange the controller attached to the headset for the one on the wall charger after each use.
- Alternating one handheld controller for another routinely throughout the day will ensure a fully charged headset for each new test.

WHEN ATTACHING A CONTROLLER TO THE HEADSET, MAKE SURE THE CORD IS FIRMLY SEATED INTO THE USB RECEPTACLES, THEN PRESS THE CENTER BUTTON TO ACTIVATE THE INDICATOR LIGHT.

If you have additional questions, contact the MacuLogix Tech Support line at 1.888.392.6801.

#### CONTROLLER INDICATOR LIGHTS

##### Solid Green = Full Power


Controller ready for use

##### Flashing Green = Low Power

Enough power for one test

##### Flashing Amber = Very Low Power

Not enough power to test

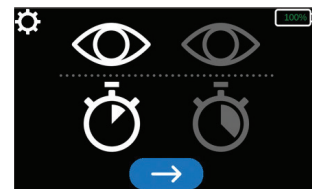

On the LCD display screen of your AdaptDx Pro, in the upper right corner there is an icon with the percentage of battery charge left in the controller.