


AdaptDx[®] Set-up Guide

I Unpacking

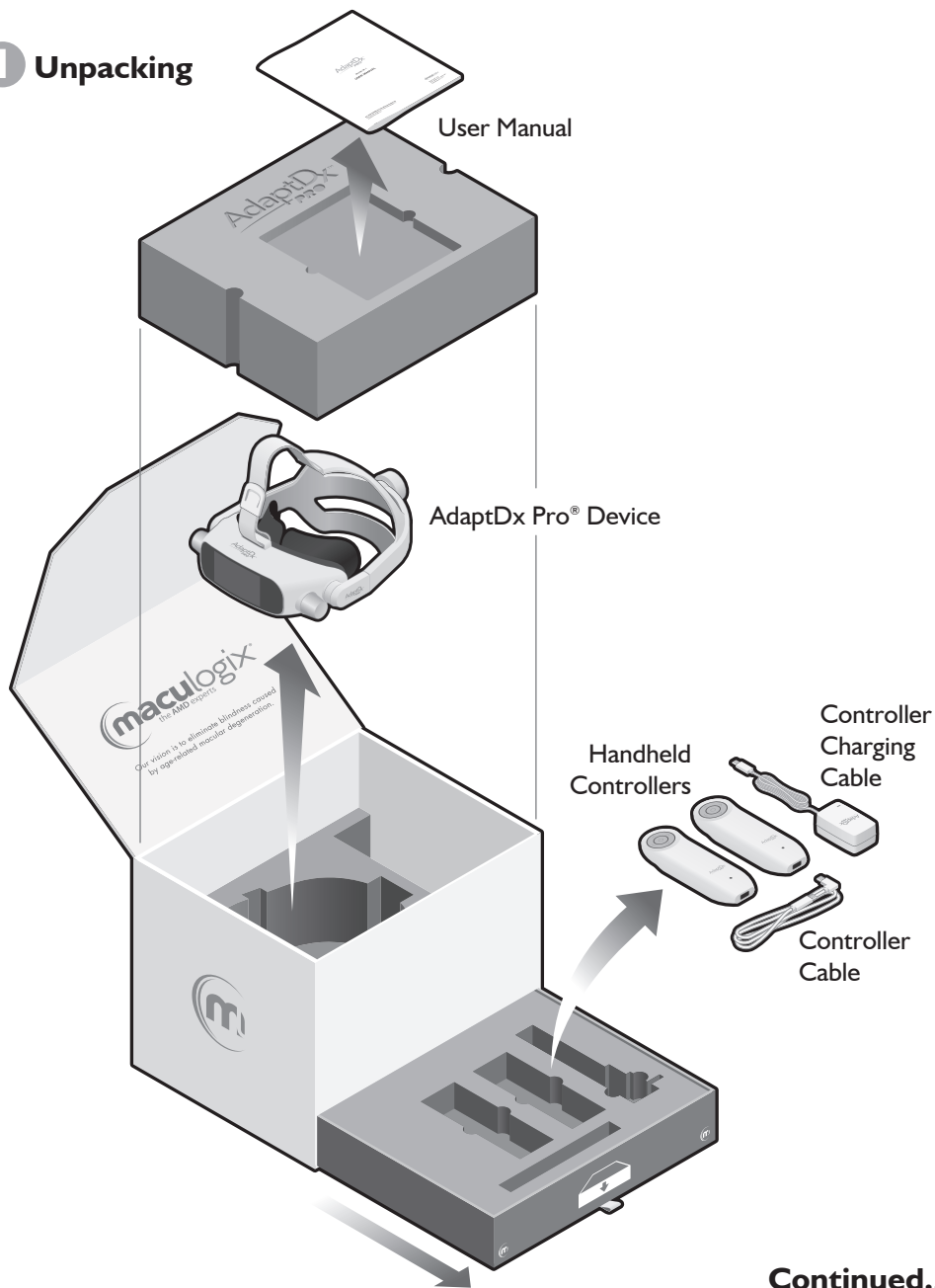


Continued...

Power supply management is vital to consistent trouble-free testing.


- The AdaptDx Pro system comes with two handheld controllers.
- Each controller has a rechargeable battery which provides power to the headset.
- Fully charge one handheld controller prior to setting up the AdaptDx Pro.
- A charged controller must be firmly connected to the headset, and remain connected at all times, even when stored.

The other controller should be left on the wall charger until swapped for the depleted one on the headset.


2 Fully Charge the Controller

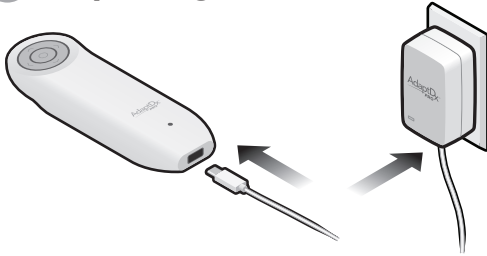


3 Connecting the Controller to the Headset

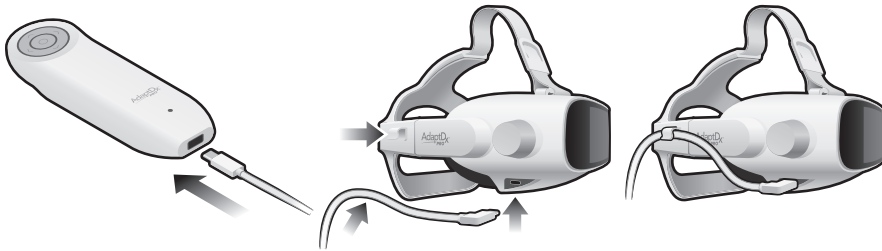

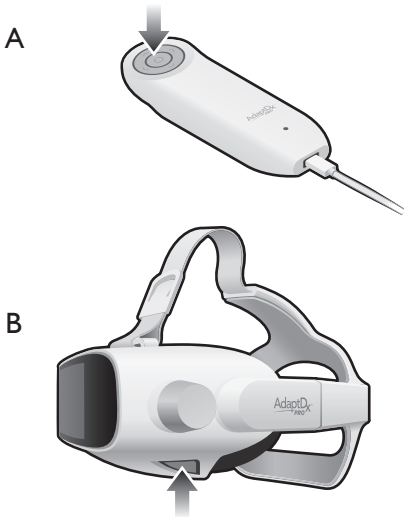

4 Powering On

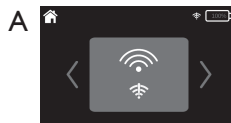


5 Setting up the WiFi




Select the “System Tools” icon  at the top of your screen. Scroll through the setup options and choose “Select Network”.


- The Wi-Fi indicator in the top right corner will be yellow  if the device is not connected.

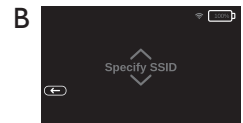

- Scroll through the available networks using the up and down arrows and choose the desired network.



- Next, enter the network password by using the up and down arrows to select each character.
- Navigate to the “Enter” button  to save the password.

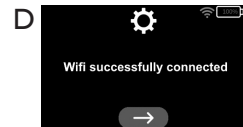

- The WiFi indicator will turn green  when the device is connected.

To order replacement parts visit store.maculogix.com or contact Customer Support at 1.888.392.6801

PN: 10644 Set-up Guide, English-US
PRO-LBL-10644 AdaptDx Pro Set-up Guide (Rev 4)
© 2021 MacuLogix, Inc.

 maculogix[®]
the AMD experts

MacuLogix[®], Inc.
1000 Kreider St. Ste 700
Middletown, PA 17057